



Phosphoric Acid Etch (Direct and Indirect)

Total Etch

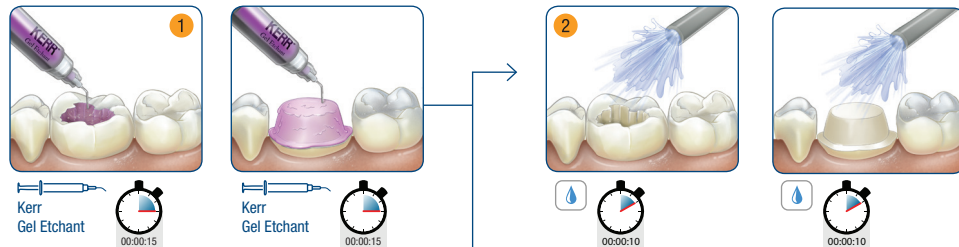

Selective Etch

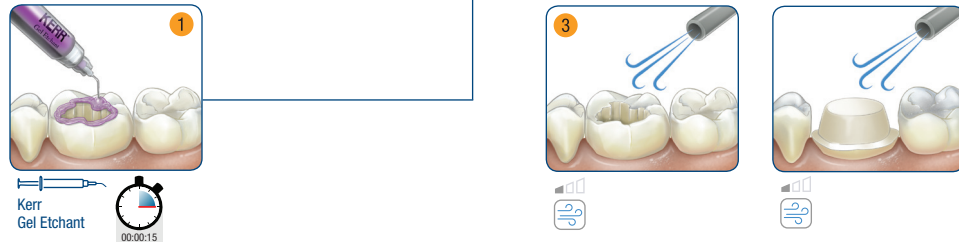

Fold

kerrdental.com

en: Refer to complete Instructions for Use regarding specifics by indication.
 fr : Reportez-vous au mode d'emploi détaillé en ce qui concerne les particularités par indication.
 es: Consulte las instrucciones de uso completas para conocer algunos aspectos específicos por indicación.
 de: Einzelheiten zu Indikationen finden Sie in der vollständigen Gebrauchsanweisung.
 it: Fare riferimento alle istruzioni per l'uso complete per quanto riguarda le specifiche per l'indicazione.
 da: Se den komplette brugsvejledning vedrørende specifikationer i forhold til indikationer.
 sv: Se komplett bruksanvisning angående specifik information per indikering.
 el: Ανατρέξτε στις οδηγίες χρήσης για διεκρινίσεις ανά ένδειξη.
 nl: Raadpleeg de complete gebruiksaanwijzing voor specifieke instructies per indicatie.
 no: Se komplett bruksanvisning vedrørende detaljer ved indikasjon.
 fi: Katso käyttöohjeita koskevat tiedot käyttöohjeista.
 pt: Consulte as instruções de utilização completas no que respeita às especificidades por indicação.
 pl: Szczegółowe informacje o wskazaniach do zastosowania znajdują się na ulocie produktu.
 ru: Особенности в отношении показаний см. в подробных инструкциях по использованию.
 ro: Consultați instrucțiunile complete de utilizare cu privire la caracteristicile pe baza indicațiilor.
 hu: Az indikációk szerinti specifikációkat tekintse meg a teljes szövegű Használati útmutatóban.
 si: Za posebnosti pri posameznih indikacijah glejte celotna navodila za uporabo.
 cs: Informace o specifikacích v závislosti na indikaci najdete v kompletním návodu k použití.

tr: Endikasyona göre ayrıntılar için Kullanım Talimatları'na başvurun.
 et: Täpsemad kasutusjuhised olenevalt otstarbest leiab juhendi täisversioonist.
 lv: Specifisku informāciju par indikācijām skatiet pilnā lietošanas instrukcijā.
 lt: Specifines savybes pagal indikacijas žr. išsamiose naudojimo instrukcijose.
 sk: Informácie o špecifikáciách v závislosti od indikácie nájdete v kompletnom návode na použitie.
 ar: ارجع إلى تعليمات الاستخدام الكاملة للاطلاع على الخصائص حسب الدواعي.
 bg: Вижте пълните инструкции за употреба за специфични действия според индикацията.
 hr: Proučite cjelovite Upute za uporabu u vezi s pojedinostima po indikaciji.
 sr: Pogledajte celokupna uputstva za upotrebu za deo koji se odnosi na određene primene.
 ja: 適用別の仕様については、取扱説明書全文を参照してください。
 ko: 용도별 세부 사항은 사용법 전체 내용을 참조하십시오.
 zh: 有关具体适应症，请参阅完整使用说明。
 zh-t: 請參閱關於具體指示的完整使用說明。
 id: Baca Petunjuk Penggunaan lengkap untuk mengetahui detail indikasinya.
 th: โปรดดูคำแนะนำการใช้งานฉบับสมบูรณ์เกี่ยวกับข้อบ่งชี้เฉพาะตามข้อบ่งชี้
 vi: Tham khảo Hướng dẫn sử dụng đầy đủ về chi tiết cụ thể theo chỉ dẫn.
 uk: Подобиці щодо показання див. у детальній інструкції з використання.


OptiBond, Unidose, NX3, Maxcem Elite, Demi, and Kerr are all trademarks owned by Kerr Corporation and are registered with the U.S. PTO and other countries.

Kerr™

Kerr™

Preparation

Unidose™

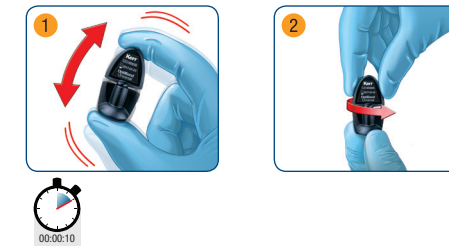

Bottle



Fold

OptiBond™ Universal ▶ Technique Guide

Direct Restorations

Unidose




Bottle



OptiBond™ Universal ▶ Technique Guide

Crowns

Unidose

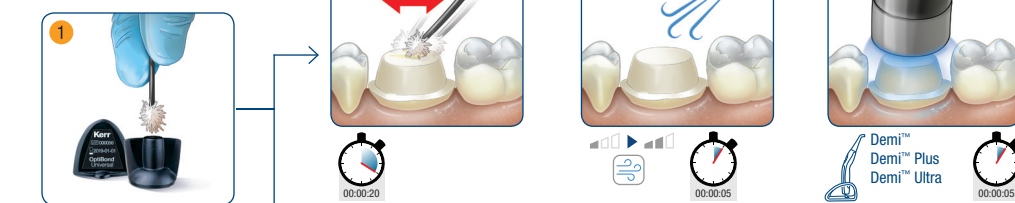


Bottle



OptiBond™ Universal ▶ Technique Guide

Kerr

Veneers

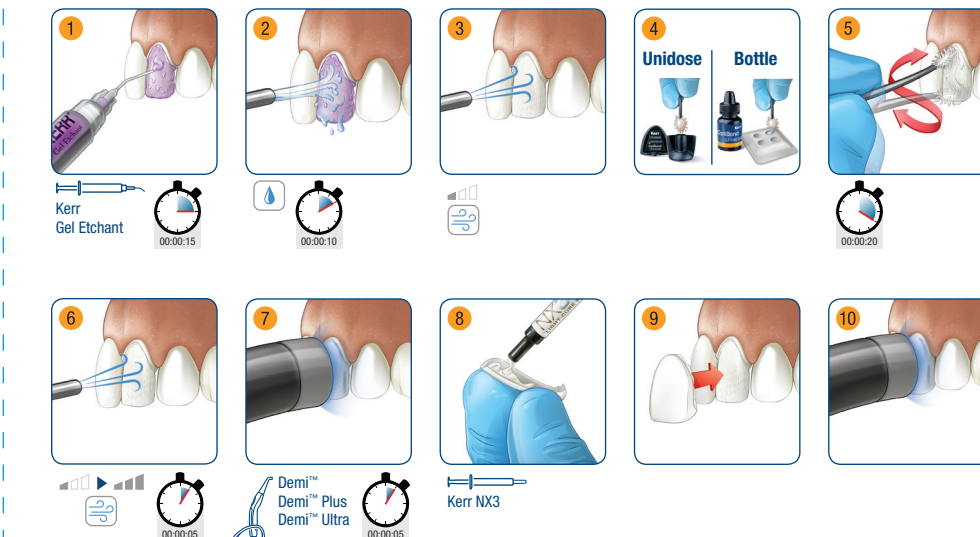



Product Name: OptiBond Universal Technique Card
Artwork Part Number(s): DMC3191_Rev.00

- 1) Binding: N/A
- 2) Corner Radius: N/A
- 3) Colors: CMYK
- 4) Size: 5.75" (W) x 5.75" (W) x 5.625" (W)
Flat Size: 17.125 (W) x 4" (H)
- 5) Material: #80 Cover
- 6) Adhesive Material: N/A
- 7) Coating: UV
- 8) Unwind Direction: N/A

Inks



CYAN



MAGENTA



YELLOW



BLACK

Revision Number: 00

Revision Date: 09/2017

Revision Summary: Product steps and Regulatory
compliance revisions

Fold Line



Last Panel Score Line

