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EN SILAN-IT is a single-component used as an adhesive primer to reinforce bonding between glass-based ceramics during fracture reparation. SILAN-IT is also indicated for the preparation of fiber-based elements during fiber-reinforced reconstructions (posts and inlay-cores).

User guide:

- 1 – Etch the surface to be treated. Rinse and dry.
- 2 – Pour a small quantity of SILAN-IT on a tray.
- 3 – Apply the product onto the previously etched and dried porcelain surface using a micro-brush and let dry.

A gentle flow of oil and water free air may be applied to accelerate the product's evaporation.

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4 – Proceed with your usual cementation procedure.

CAUTION: In case of intraoral procedure, isolate the area to be treated with a rubber dam. Cautiously close the bottle after each use for a better conservation as the product is volatile.

Composition: Ethanol (96%) - Silane (4%)
Packaging: 1 bottle (5ml)

Storage: Do not store above 24°C. Recap after use and keep refrigerated to optimize freshness of the product.

Expiration date: two years after manufacturing. Do not use after expiration date.

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