Dentoclic

FR

COMPOSITION

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EN**COMPOSITION**

Dentoclic posts are manufactured from stainless steel (316L, 1.4404) and titanium alloy (grade 5, TA6V titanium).

INDICATION

The metallic posts and keys are designed to support temporary or permanent prosthetic reconstruction (direct or indirect method) following root canal treatment. The burnout posts and keys are designed to allow the prosthetist to make a post and core. The drills are designed to prepare the space for the posts and keys.

CONTRAINDICATIONS

Do not use burn-out posts for taking impressions. Do not use burn-out posts for direct Corono-Radicular reconstruction.

Side effects

The products contained in this kit do not cause any side effects.

Unwanted effects

To avoid risks of infection, follow the advice regarding sterilisation. Reusing the posts can cause cross-contamination.

PRECAUTIONS BEFORE USE

Dentoclic stainless steel and titanium posts must be sterilised before use. Dentoclic drills must be sterilised before the first use and after each subsequent use. Warning: Dentoclic burn-out posts and main boxes cannot be sterilised. Use of the dam is recommended when this is possible and deemed necessary. Do not use burn-out posts for taking impressions. There is a difference of 2/100 between the metallic posts and the burn-out posts; the burn-outs are thinner in order to allow for the thickness of the sealing cement and prevent excessive friction between metal and tooth. It is therefore essential to use Dentoclic metallic posts with the associated Dentoclic burn-out posts. Be careful for the difference between the lengths and diameters of the posts. Remove the colour code using a suitable tool before inserting the post. Do not reuse single-use devices (Itena posts and screw-posts). Drills are designed for a maximum use of 15 cycles.

Dentoclic drills are designed for use with classic slow-speed contra-angle devices, without exceeding 10,000 rpm. Neither the Dentoclic burn-out resin posts nor the main box may be sterilised.

HYGIENE AND STERILISATION

Sterilisation of the posts before their first use or before each reuse is the responsibility of the user. If the user makes use of dirty and/or damaged instruments, Itena Clinical cannot be held liable. Clean the instruments as quickly as possible after use. Do not use detergent solution with an acid, abrasive or chlorine base.

Manual cleaning:

Clean the instruments with a suitable nylon brush in order to remove smear. Immerse the instruments fully for 15 minutes in a disinfectant solution suitable for medical devices. Check to ensure the absence of dirt on the instruments. If necessary, repeat the cleaning and disinfection cycle. Rinse the instruments thoroughly with de-mineralised or distilled water to avoid corrosion and alteration during the sterilisation process. Dry with compressed air for 2 minutes. Check visually for cleanliness and discard any instruments that may be distorted. All metallic parts can be cleaned with ultrasound and sterilised in an autoclave using our sterilisable kit at 134°C for 18 minutes (maintenance time).

Automatic cleaning:

Use a cleaning and disinfection device, sorting the instruments according to type and category of material, in order to avoid electrolysis and corrosion. Arrange the instruments so that they are immersed fully and start the cleaning cycle according to the following instructions:

Stages	Type of product and concentration	Time / Temperature
Pre-wash	/	/
Washing	Disinfectant solution suitable for medical devices*	15 minutes at 55°C
Rinsing	Demineralised water	2 minutes at ambient T° (15-25°C)
Drying	/	According to autoclave parameters

At the end of the cycle, check that no dirt remains on the instruments. Repeat the cycle if necessary.

STERILISATION

Place the instruments that can be sterilised in suitable sterilisation bags (according to EN ISO 11607). Place the sterilisation bag in the autoclave, then carry out a sterilisation cycle: 134°C at 2.2 bars for 18 minutes.

Store the instruments well away from sources of contamination.

INSTRUCTIONS FOR USE

Prepare the root canal using the following:

1. The Gates/Largo drill, which removes the gutta-percha (A)
2. The Dentoclic «Pilot» drills: help widen the canal.
3. The Dentoclic Conical Drills or Cylindrical-Conical Drills: prepare the holding for the root post according to the practitioner's choice (B)

Check that the root is whole before inserting the post.

For greater safety, use increased drill diameters in succession in order to widen the canal progressively to the desired diameter.

Ensure that sufficient dentine wall is preserved (C)

2 possible methods:

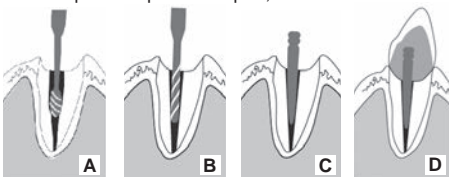
1) Indirect method:

Insert the metallic post to take the impression. Send the impression to the laboratory with the post and attach the corresponding burn-out post.

If there are divergent roots, attach the corresponding Dentoclic key(s).

Insert a stainless steel post of the same diameter into the canal and make a temporary crown. Only place the temporary cement on the edges of the crown, not on the post.

On receipt of the prosthetic part, test it and seal to allow final restoration. (D)

2) Direct method:

Insert the metallic titanium post in the canal and seal definitively.

Make the temporary or permanent crown.

TARGETED USERS

Products contained in Dentoclic boxes are designed for use by dental professionals only (dental surgeons and prosthetists).

CONDITIONS OF STORAGE

Burnout posts and keys: no specific conditions of storage. Metallic posts: store away from damp.

DISPOSAL OF PRODUCTS

Used products must be disposed of with waste that carries an infection risk.

For all incidents involving the product, contact Itena Clinical and the competent authority of the country in which the incident occurred, as soon as possible.

CYLINDRO-CONICAL				
	Stainless steel		Burn-out	
	Diameter	Length	Diameter	Length
White	1.2 mm	9.6 mm	1.15 mm	9.5 mm
Long White	1.2 mm	11.5 mm	1.15 mm	11.4 mm
Yellow	1.3 mm	9.6 mm	1.25 mm	9.5 mm
Long Yellow	1.3 mm	11.5 mm	1.25 mm	11.4 mm
Red	1.4 mm	11.6 mm	1.35 mm	11.5 mm
Long Red	1.4 mm	13.5 mm	1.35 mm	13.4 mm
Blue	1.5 mm	13.6 mm	1.45 mm	13.5 mm
Green	1.6 mm	15.6 mm	1.55 mm	15.5 mm
Black	1.7 mm	17.6 mm	1.65 mm	17.5 mm

CONICAL				
	Stainless steel		Burn-out	
	Diameter	Length	Diameter	Length
Yellow	1.52 mm	9 mm	1.5 mm	9 mm
Orange	1.72 mm	11 mm	1.7 mm	11 mm
Green	1.8 mm	12 mm	1.78 mm	12 mm
Red	1.82 mm	13 mm	1.8 mm	13 mm
Blue	2.02 mm	14 mm	2 mm	14 mm
Purple	2.22 mm	16 mm	2.2 mm	16 mm

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