Vinylsiloxanether® impression material
DIN EN ISO 4823


KETTENBACH DENTAL
Simply intelligent

VSXE® ONE

INSTRUCTIONS FOR USE / Gebrauchsinformation / Mode d'emploi / Gebruiksaanwijzing /
Инструкция по использованию / Istruzioni per l'uso / Instrucciones de uso / Kullanım Kılavuzu /
Instruções de uso / Instrukcja użycia / Інструкція для застосування / 取扱説明書 / تعليمات الاستخدام

CONTENT

Inhalt / Contenu / Inhoud / содержимое /
Contenuto / Contenido / İçerik / Conteúdo /
Treść / Зміст / 目次 / الفهرس

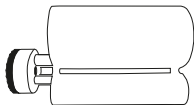Deutsch	4
English	10
Français	16
Nederlands	23
Русский	30
Italiano	37
Español	44
Türkçe	51
Português	58
Polski	65
Українська	73
日本語	82
89	عربي

Vinylsiloxanether® impression material
DIN EN ISO 4823

VSXE® ONE
MONOPHASE

For dental professional use only.

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Product Description

VSXE® One is an elastomeric precision impression material with vinyl polysiloxane and polyether components (Vinylsiloxan-ether®).

VSXE® One has medium viscosity and is available in 380-mL cartridges, 5:1, for use in the usual automatic dosing and mixing systems, e.g. Sympress.

Indications / Techniques

VSXE® One is to be used as a medium bodied tray or syringeable impression material in one-step technique (monophase) for:

- Implant impressions
- Fixation impressions
- Functional impressions
- Impressions for crowns/bridges, inlays/onlays and veneer preparations

Patient Target Group

Patients being treated in the context of a dental procedure.

Warnings

Do not use VSXE® One for two-step impressions (two-stage impression technique) or temporary relinings.

Do not mix or use VSXE® One with other elastomeric impression materials.

Potential Allergenic Substances

Platinum compound, 4-methoxyphenol.

Cautions

Do not use after expiration date.

Normally, impression materials do not cause allergic reactions; however, an allergic reaction cannot be ruled out in sensitive individuals.

If in doubt, we recommend performing an allergy test before use.

Do not use the material in case of known allergies to any of the ingredients or in case of contact allergies.

Do not leave any residual impression material in the sulcus or oral cavity.

Do not swallow impression material! If swallowed: Consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Use of products containing active sulfur, aluminium chloride or nitrogen compounds (retraction cords containing ferric sulfate, polysulfide impression materials, etc.) in conjunction with this product will interfere with the setting reaction of the Vinylsiloxanether[®] material. Use of these materials requires the area to be rinsed thoroughly to remove all residue. Avoid contact with latex gloves.

When taking impressions of areas with severe undercuts and wide interdental spaces, standard blocking-out techniques must be used.

When using a custom impression tray, ensure that sufficient space remains between the side of the tray and the teeth/jaw.

Do not allow the material to enter the sewer or water system to avoid environmental hazard.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Impression tray: Preparation and Adhesives

In principle, all common impression trays can be used if a relevant dynamic pressure is guaranteed. When retention is not sufficient, to ensure a strong bond to the impression material, brush the impression tray with Universal Adhesive prior to

loading the tray with impression material. Allow to dry as per instructions for use and cover with a sufficient amount of impression material.

Instructions for the Safe Handling of the Kettenbach 380-mL cartridge system; 5:1

Components: 380-mL cartridge, Dynamic Mixers

Prior to first use please unscrew the blue plug counter-clockwise and pull it out (Figure 1, Page 96). Check the outlet opening. Then screw on the dynamic mixer (Figure 2, Page 96). Ensure that the positioning aid is on the top.

Please use only the dispensing and mixing components compatible with the Kettenbach 380 ml cartridge system for dispensing and mixing impression material.

The Kettenbach 380 ml cartridge is heavy and could be damaged if dropped. Damaged cartridges must not be used under any circumstances.

When using the material in a Sympress II dispensing device, please activate the speed setting indicated on the product label (Figure 6, Page 97). Failure to comply with this instruction may modify the curing characteristics of the impression materials.

If you are using a unit other than the Sympress mixing device, please follow the instructions for the automatic dispensing and mixing system you are using.

When changing to a different mixing device, always discard at least 3 cm of mixed material.

Using the Kettenbach 380-mL cartridge system; 5:1 in Dispensing Device Sympress / Sympress II

By pressing the plunger return knob, allow the plunger to return to its original position automatically (while the cover is closed; Figure 3, Page 96). Next, open the cover (Figure 4, Page 96).

Insert the Kettenbach 380 ml cartridge with the pre-assembled mixer in the Sympress/Sympress II dispensing and mixing unit (Figure 5, Page 97). If you are working with a previously used cartridge, the filled mixer will already be located on the cartridge as a cap.

If you are using a Sympress II dispensing device, please activate the speed setting indicated on the product label. A light signal will confirm your selection (Figure 6, Page 97).

Close the device cover (Figure 7, Page 97).

Using the Cartridge for the First Time

Dispense the impression material (ensure that the dynamic mixer is wound into place fully) until the base material and catalyst are mixed homogeneously; discard at least the first 3 cm when dispensing (Figure 8, Page 97).

At least 3 cm of the mixed material must be discarded each time the plunger is restarted or when using the cartridge in different mixing units.

Load the impression tray (or application syringe) with the required amount of material (Figure 9, Page 97) by pressing the appropriate feed key. Hold the impression at an angle when loading and press against it lightly. Leave the mixer in the material when dispensing to avoid air bubbles.

Release the feed key as soon as the required amount of material has been dispensed. The filled dynamic mixer should remain on the cartridge as a seal.

Remove the used mixer before the next application by turning it to the left (Figure 10, Page 97), visually check the outlet opening for blockage and attach a new dynamic mixer (Figure 10, Page 97).

Further Use of a Cartridge Already in Use

Wind a new dynamic mixer onto the same mixing unit and begin the new application (material no longer has to be discarded).

Product Description Applyfix 5

Applyfix 5 application syringe is useful for intraoral application of elastomeric impression materials, especially for material dispensed from the Kettenbach 380-mL cartridge system or from other automatic mixing systems.

Contents Applyfix 5 (Figure 1, Page 99)

1. Applyfix 5 syringe tips
2. Tip retainer to attach syringe tips
3. Syringe cylinder
4. Syringe piston
5. Cleaning brush
6. Ring washer

Application instructions Applyfix 5

Insert a new syringe tip into tip retainer. Screw tip retainer with the syringe tip onto the syringe cylinder (Figure 2, Page 98).

If necessary, lubricate the ring washer very slightly with petrolatum for an easy action of the syringe piston.

Prepare the impression material (the Applyfix 5 application syringe is indicated for light bodied or medium bodied consistency impression material only).

Attach the open syringe cylinder of the Applyfix 5 directly to the mixing tip of the preferred impression material (Figure 3, Page 98).

Fill the application syringe with the required amount of impression material by pressing the feed button (Fig. 9, page 97). Insert the syringe piston into the syringe cylinder (Figure 4, Page 98) and apply the impression material into the patient's mouth as desired.

Cleaning and Disinfection of the Applyfix 5

Clean the Applyfix 5 application syringe with the brush (Figure 5, Page 98) after using and removing of the full hardened impression material. The Applyfix 5 application syringe can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions. The Applyfix 5 can be reused and autoclaved several times and should be replaced once it shows signs of mechanical damage. Do not reuse the syringe tips, to avoid contamination.

Cautions Applyfix 5

Only use one material per Applyfix 5 application syringe. Contamination of different impression materials renders the impression material unusable.

Storage Conditions

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

Disinfection

Impressions can be disinfected using for example a 2 % acidic glutaraldehyde solution. Use disinfection solutions specific for impression materials. See manufacturer instructions.

Model Preparation

Impressions with VSXE® One may be poured immediately after disinfection of the impression up to several weeks with high quality dental stone (class III–V). Impressions can also be poured several times.

Composition

Vinyl polysiloxanes, Hydrogen polysiloxanes, Filler, Pigments, Additives, Platinum compound.

Disposal

Completely emptied cartridges / Dynamic Mixers / applicator tips and set material can be disposed in the household rubbish.

Trademarks

Vinylsiloxanether® is a registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials are available in certain markets only.

For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: March 14, 2023

Product Overview

Product name	ISO 4823	Consistency (approx.) mm	Mixing ratio and total content volume per unit	Mixing technique	Mixing element
VSXE® One	Type 2 Medium bodied	38	5:1, 380 ml cartridge	Sympress/Sympress II or other automatic dispensing and mixing unit	Dynamic mixer, blue

Technical Data

Product name	Working time at 23°C/74°F ≤	Working time at 35°C/95°F ≤ ¹⁾	Intraoral setting time at 35°C/95°F ≥	Total setting time* ≥
VSXE® One	2 minute 30 seconds	1 minute 20 seconds	2 minutes 45 seconds	5 minutes 15 seconds

* Total setting time (removal time from mouth) from start of mix

Note: To ensure optimal impressions, the temperature of the material should not deviate from 23°C (74°F) before applying. Otherwise, working and setting times will be affected.

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