## INSTRUCTIONS FOR USE

**INTENDED USE:**

Essix TRAY-RITE Plastic is used to fabricate intra-oral tooth whitening (bleaching) trays.

**INDICATIONS FOR USE:**

Essix TRAY-RITE Plastic is indicated for a patient undergoing a tooth whitening procedure.

**INTENDED USER:**

Intended to be used in a manufacturing setting within a dental laboratory, dental office or medical device manufacturing facility to fabricate a removable, custom-made intra-oral dental appliance.

**PERFORMANCE CHARACTERISTICS AND CLINICAL BENEFITS:**

Performance characteristics of orthodontic plastics include a good combination of strength, durability and clarity, resistant to cracking or breaking, easily removeable from models and effortless to trim.

Clinical benefits of tooth whitening and aligner treatment may include overall improvement of tooth health and correction of malocclusion(s).

**CONTRAINDICATIONS:**

This product is contraindicated for patients with a history of allergic reactions to plastics.

**WARNINGS:**

Excessive heating time during thermoforming will cause plastic to over sag and potential for injury to operator.

Improper thermoforming of the plastic appliance may cause the appliance to crack/break, resulting in sharp edges, loose pieces and possible aspiration of pieces.

Report serious incidents that have occurred in relation to the device to device manufacturer and the Competent Authority in Member State of residence, or other applicable regulatory agency.

**PRECAUTIONS:**

Patients with a history of allergic reactions to plastics should not use this product.

Do not reuse plastic sheet. Once thermoformed it cannot be reused.

Thermoformed device is for Single Patient Use. Risks of reuse include infection and contamination (i.e. bacterial, viral).

**ADVERSE REACTIONS:**

Allergic reactions.

**STEP BY STEP INSTRUCTIONS:**

Thermoforming Instructions (for complete instructions, refer to thermoforming equipment manual):

1. All models should be trimmed to 3/4" high. Full arch models should have the palate trimmed away.
2. Spray the model with Trim-Rite® Release Agent to prevent the material from sticking to the model.
3. Place the plastic into the frame of the machine soft side up and the model on the base plate.
4. Scan/Enter the code per thermo-forming machine used. Swing the heating element over the plastic and press the appropriate start button.
5. Move the heating element away from the plastic when the heating and cooling time expires.
6. Druformat: hold the element away and press the button on the side to engage the pressure chamber. Biostar: flip the heated plastic over the model.
7. After the cooling stage, release the pressure per thermo-forming machine used.
8. Immediately spray the plastic encased model with Essix Freeze Spray® Coolant. This is an essential step. Not only does it cool the plastic prohibiting further thinning, but it "shrinks" the material around the cast, realizing a tighter fit.
9. Remove the formed tray from the frame of the machine.
10. Trim and finish appliance as desired.

**\*NOTE:** The heating times below should be used as a reference as heating times may vary depending upon machine usage. If plastic does not adapt to model, add heating time until adaptation occurs. If plastic "folds" when thermoformed, reduce heating time until material does not fold. Performance with other machines have not been validated.

Thickness	Biostar™ Code	Druformat
0.5mm (.020")	123	0:55

**SAFE DISPOSAL:**

Unused plastic sheets can be recycled.

Used dental appliances may be considered a biohazard. Follow your country-specific laws, directives, standards and guidelines for the disposal of used devices.

 Manufacturer: **Dentsply Sirona Orthodontics Inc.**  
7290 26th Court East  
Sarasota, FL 34243 USA

 [www.dentsplysirona.com/ifu](http://www.dentsplysirona.com/ifu)

MKT-DFU0038-ML Rev. 03 Issued 12-2024

Biostar and Druformat are not trademarks of Dentsply Sirona.

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©2024 Dentsply Sirona