


DENTO-VIRACTIS INSTRUGERM 50

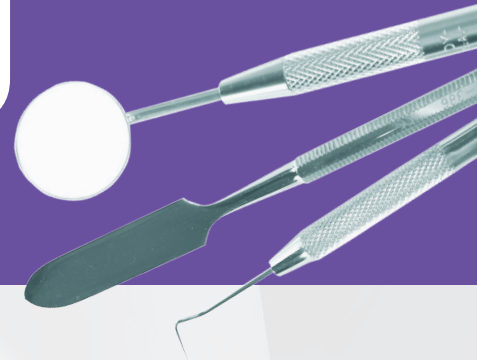

Pre-disinfection of lightly contaminated dental instruments



Dilute at 0.5% (one 5ml dose per liter of water or 1 pump for 3L)

STEP 1



Immerse the instruments for 15 minutes for full microbiological efficacy

STEP 2



Brush in order to eliminate the residue if needed

STEP 3



Rinse the instruments best with distilled water or tap water

STEP 4



Dry with a clean disposable towel

STEP 5



Insert instruments in a paper sterilising pouch, then sterilise

STEP 6



ADVANTAGES

- Kills bacteria, viruses, moistures and biofilm
- **Double action:** detergent and disinfection
- Enhanced cleaning properties due to the alkaline pH
- **Cost-saving:** concentrated at 0.5% (1L = 200L & 5L = 1000L solution)
- Aldehyde-free (no emission, no smell)
- Compatible with ultrasonic cleaners

CAUTION

- **Renew the cleaning bath every day**
- Product for external use only. Do not ingest
- Precautions for use: refer to the medical safety data sheet and the label.

Product classified according to the European Directive "dangerous preparations" 1999/45/EC.

MICROBIOLOGICAL PROPERTIES

- **Bactericidal:** EN 13727
- **Fungicidal:** EN 13624 (Candida Albicans)
- **Virucidal:** Norovirus murin, BVDV (surrogate of Hepatitis C), Herpes simplex virus type 1, PRV (surrogate of Hepatitis B), HIV

DV50-PEN-V1