



TEMFIL II

Ready-to-use paste.

Eugenol-free calcium sulphate zinc oxide resin.

Application:

- For the temporary filling of cavities.



Why choose R&S Temfil II?

- + Elastic consistency**
Ensures easy use.
- + Strong**
Excellent resistance to compression.
- + Safe**
Does not cause pulp irritations. Controlled setting expansion.
- + Sealant**
Adheres perfectly to dental tissues to form a hermetic seal.
- + Easy separation**
Easy removal using an excavator or broach, before replacing with permanent material.



Advice from our professional dentists

- I recommend this product because of its easy-to-use consistency.
- Store in a cool and dry place (5-25°C). It is important to replace the lid after each use.
- I also use it for inter-appointment intracanal medication.
- Once I have finished using **TEMFIL II**, I ask my patients not to apply any chewing pressure for the following 15 minutes.

Technical data and use



TEMFIL II

Colour

White

Resistance to compression (MPa)

25

pH

6

Working time

15 min*, maximum

Setting time in mouth

4 min*.

Odour

Odourless

Composition

D.M.T.G (glycol) < 12%
Polymethyl methacrylate < 3%
Calcium sulphate < 58%
Zinc oxide < 11%
Barium sulphate < 17%
Amorphous silica < 2%

Solubility

Insoluble in water

Use

Using an appropriate filling instrument, put Temfil II in the cavity and adapt as required.
It is not essential to dry the cavity before filling it.
Temfil II hardens through moisture absorption to provide an impermeable seal.

Packaging

30-g tube

Lifespan (in years)

2

Reference

12-464


* Time observed at a temperature of 23°C. For higher temperatures reduce the time, for lower temperatures increase the time.

Additional products

Mouth spatulas



Diamond burs

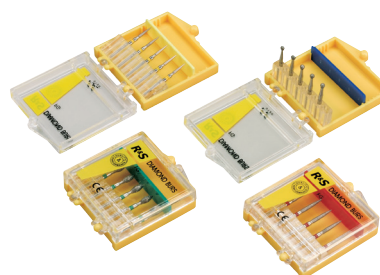

Temfil II and burs: class IIa medical devices / CE 0120 Marking / Certifying body: SGS UK LIMITED.
Mouth spatulas: class I medical devices / CE Marking. Images are for illustrative purposes only.
Manufactured by CFPM.

GACD - 25 rue Bleue 75009 PARIS - France 435 051 776 R.C.S PARIS - SASU.
Tel.: +33 (0)1 48 01 27 14 - Fax: +33 (0)1 45 23 51 89.
export@rs-dental-products.com - www.rs-dental-products.com.