

# Pola Luminare

SDI

polaLuminare

Brush-on tooth whitening system

(6% Hydrogen Peroxide)

INSTRUCTIONS FOR USE

Pola Luminare is a pleasantly flavoured, 6% hydrogen peroxide whitening solution. Designed to easily brush directly onto teeth to form a thin whitening film without the need of trays. Before using Pola Luminare, your dentist should assess your suitability for the product including your medical history and explain the application procedure.

**INDICATIONS FOR USE:**

- Whitening of discolored teeth

**INSTRUCTIONS:**

1. Brush and floss teeth as normal.
2. Blot dry the front surface of teeth.
3. Whilst retracting the lips, apply a thin layer of Pola Luminare solution to the front surfaces of the teeth, re-loading brush as required.
4. After application do not let your lips touch your teeth for 30 seconds.
5. Leave Pola Luminare solution on teeth for 30 minutes. Do not eat or drink during this time to maximize results.
6. At the end of 30 minutes, remove the Pola Luminare solution by brushing with a tooth brush.
7. Use Pola Luminare twice a day for 14 days.

For best results, one of the daily treatments should be applied before going to bed, allowing the Pola Luminare to stay on overnight. When not in use, firmly replace cap to prevent solution from drying out.


**STORAGE AND HANDLING:**

- Store in a cool and out of direct sunlight at temperatures between (2°-25°C / 35°-77°F).
- Do not store in clothing pockets.

E  
N  
G  
L  
I  
S  
H

**PRECAUTIONS:**

- Irritant. Avoid contact with eyes or gums. Avoid swallowing.
- To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
- Not to be used by pregnant, lactating women or person under 18 years old.
- Avoid contact with eyes, rinse immediately if product comes into contact with them.
- Use at room temperature.
- Pola Luminare should not be used by patients with known allergies or chemical sensitivity to peroxides, glycerol etc.
- Do not smoke during or immediately after the whitening process.
- Keep product away from heat and naked flame, as product is flammable.
- Discard Pola Luminare after treatment has been completed.
- SDS available at [www.sdi.com.au](http://www.sdi.com.au) or contact your regional representative.
- Contains Sodium Fluoride and contains Hydrogen Peroxide.
- Do not use after expiry date.
- Keep out of reach of children and pets.

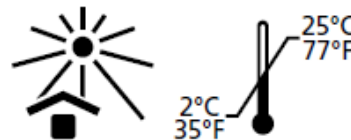

SDI